




Another choice in the trusted AnchorFast product family.

The AnchorFast Guard Select™ oral endotracheal tube fastener provides a convenient means to hold an oral endotracheal (ET) tube (with or without subglottic suction lumen[†]) securely in place without the use of adhesive tape.

In a clinical study*, clinicians found the protective sleeve's ability to maintain the main airway by preventing occlusion of the ET tube acceptable for 100% of patients evaluated.

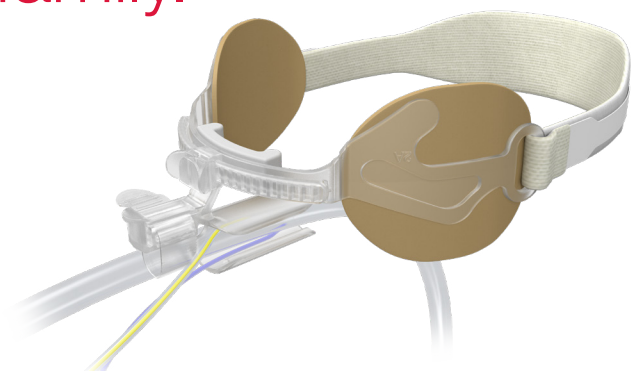

AnchorFast Guard Select
Oral Endotracheal Tube Fastener

AnchorFast Guard Select

Oral Endotracheal Tube Fastener

In 90% (27 of 30) of patients evaluated, clinicians stated it was **easy to place/insert all three tubes** (ET, pilot line, and subglottic lumen) into the protective sleeve.



In 86.7% (25 of 30) of patients evaluated, clinicians stated the device was **easy to shuttle** from side to side during the course of use.



 In 100% of 30 patients evaluated, clinicians found the subglottic suctioning acceptable.


In 97% (29 of 30) of patients evaluated, based on the clinicians' overall experience with the study product, **clinicians would like to see the study product used at their facility.**



*Data on file. Clinical Study 5855-I, Multiple ICU settings, n= 30 patients using ET tubes with integrated subglottic suction capability

Gliding tube shuttle allows the ET tube to be easily repositioned from side to side while the tube remains secure.

Cheek barrier pads are the primary means by which the device is secured to the patient's face.

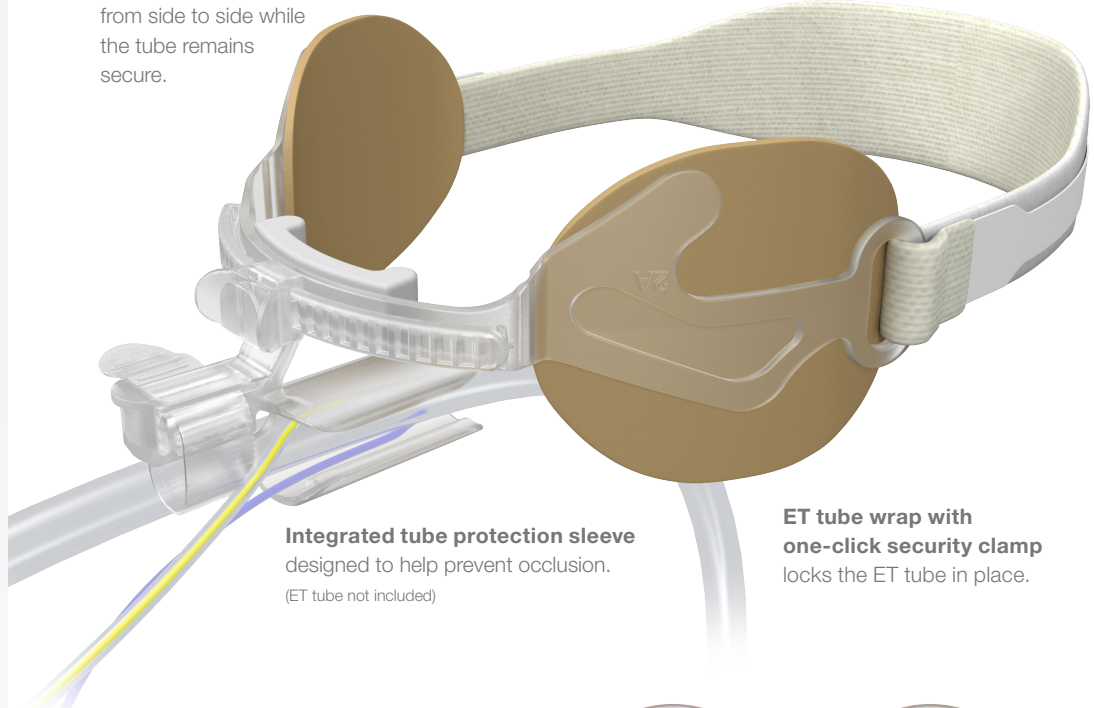

Integrated tube protection sleeve designed to help prevent occlusion.
(ET tube not included)

ET tube wrap with one-click security clamp locks the ET tube in place.

AnchorFast Guard Select incorporates all the features of AnchorFast Guard with a modified tube protection sleeve designed to accommodate both standard and ET tubes with integrated subglottic suction lumen.



AnchorFast Guard Select*



AnchorFast Guard*

Product/Description	Stock No	Quantity/Box
*AnchorFast Guard Select oral endotracheal tube fastener Secures ET tubes ranging from 5.0–9.0 mm, subglottic suctioning from 6.0–8.0 mm	9700	12
*AnchorFast Guard oral endotracheal tube fastener Secures ET tubes ranging from 5.0–10.0 mm	9800	12

For more information or to request a free sample, visit hollister.com/anchorfast



Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.800.323.4060

www.hollister.com

USA Rx Only. Caution: Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product. Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.

The Hollister logo, AnchorFast, AnchorFast Guard and AnchorFast Guard Select are trademarks of Hollister Incorporated. ©2019 Hollister Incorporated.

923370-0819

